

# How to use EpiPen® and EpiPen® Jr Auto-injectors.

Remove EpiPen® Auto-injector from carrier tube

1



- Hold firmly with orange tip pointing downward
- Remove blue safety release

2



- Swing and push orange tip firmly into mid-outer thigh until you hear a 'click'
- Hold on thigh for several seconds



#### Built-in needle protection

- When the EpiPen® Auto-injector is removed, the orange needle cover automatically extends to cover the injection needle



***After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911.***

For more information go to [www.EpiPen.ca](http://www.EpiPen.ca)

EpiPen® and EpiPen® Jr Auto-injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions.

This product may not be right for you. Always read and follow the product label.